dry-go





Electronic mini drying station for hearing aids.

INSTRUCTION MANUAL

EN	INSTRUCTION MANUAL
DE	BEDIENUNGSANLEITUNG
FR	MANUEL D'UTILISATION
ES	MANUAL DE INSTRUCCIONES
PT	MANUAL DE INSTRUÇÃO
IT	MANUALE D'USO
NL	HANDLEIDING
TR	MANUALE D'ISTRUZIONI
DK	BETJENINGSVEJLEDNING

FI	KÄYTTÖOHJEET
SE	BRUKSANVISNING
NO	INSTRUKSJONSHÅNDBOK
HU	HASZNÁLATI UTASÍTÁS
PL	INSTRUKCJA OBSŁUGI
CZ	UŽIVATELSKÁ PŘÍRUČKA
SL	NAVODILA ZA UPORABO
BG	РЪКОВОДСТВО ЗА ЕКСПЛОАТАЦИЯ
RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ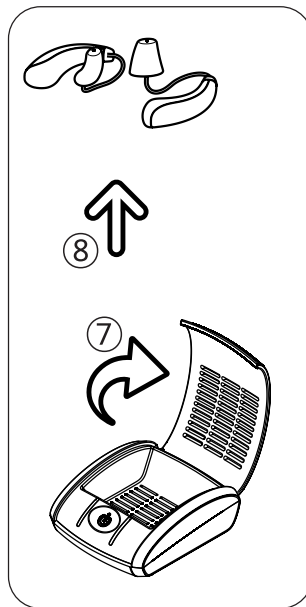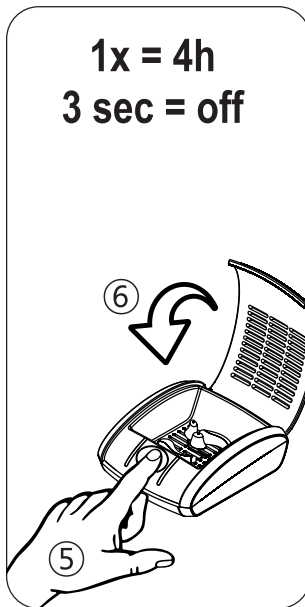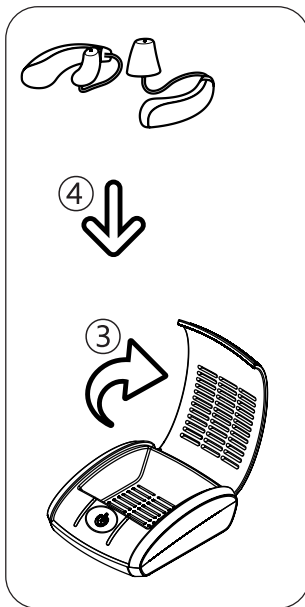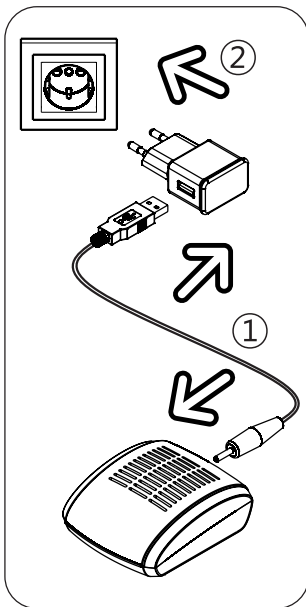

Dries and cares your hearing systems

Moisture and ear wax can impair the function of hearing systems and cause failures.

The dry-go offers optimal care. Comfort drying at 50°C extends life time significantly.

Using first time

1. Insert the USB-cable into the 5 V power supply and the round plug connector into the jack on the rear side of your drying station
2. Then plug the power supply into an AC power socket

① The dry-go is ready to use

Operation

3. Open the lid of your dry-go
4. Place the with wet wipes pre-cleaned hearing systems with the battery-door open (if possible) into your drying station
5. Briefly touch the sensor button and the comfort drying program is starting automatically
 - The 4 hours drying program with approx. 50°C warm air is fully processor controlled
6. Close the lid
 - After 4 hours the drying station automatically shutdown to stand-by
7. Open the lid again
8. Remove the dried and hygienically cleaned hearing systems

① LED permanent light

① LED switch off

Maintenance and operation

- We recommend wet wipes for cleaning your dry-go
- You can interrupt the drying process by holding the sensor button for more than 3 seconds
- The dry-go is very energy efficient; unplug the power supply only for prolonged absence

Warnings

- Avoid contact with water
- Don't use the dry-go near basins etc.
- In the event of a malfunction, unplug the power supply and take the drying station to your retailer
- Don't disassemble the drying station and use only the supplied 5 V power adapter and USB-cable
- Children must be supervised to ensure that they don't use the drying station as a toy
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station
- The drying station has an automatic room and temperature modulation as well as safety shutdowns

- Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities
- Contact your local government for information regarding the collection systems available
- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being

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